


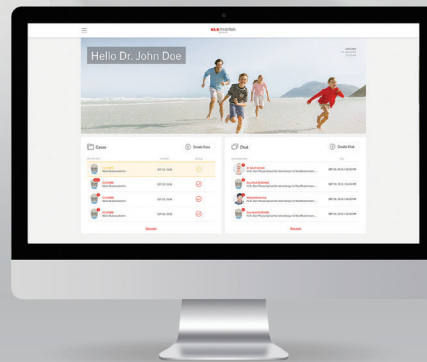

IPS® Gate Onboarding

Individual Patient Solutions

every one is unique

www.klsmartinnorthamerica.com

Products shown may not be licensed or cleared in all markets.
Availability is subject to change without notice. Licensed for US and CA.



SCAN ME
To Download
the IPS® Gate App

Registration

New IPS® Users

- Navigate to <https://ips.klsmartin.com>.
- Click on the '*Not a Member Yet?*' link and provide the requested information.

Existing IPS® Users




- An activation email will be sent to the user's address.
- Click the link to receive a temporary password.
- Log in with email and temporary password at <https://ips.klsmartin.com>.
- Agree to terms and conditions, reset password and setup MFA.

Surgical Plan Approval

- If the App is installed, a push notification will be sent when the plan is ready for approval.
- Emails are also triggered.
- Follow the hyperlink right to the case and select '*Surgeon Approval*'.
- Approve or request a redesign and enter your password.
 - ◊ If requesting a redesign, let us know the changes you would like in the chat.



For a complete user guide,
scan this QR Code

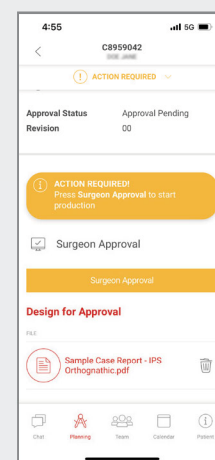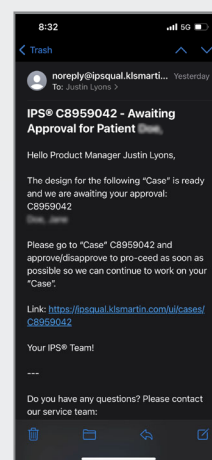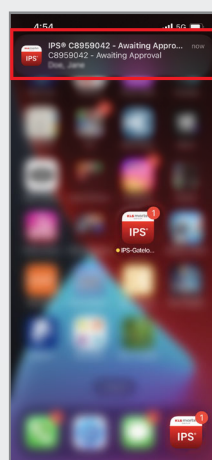

KLS-Martin L.P.

Mail: P.O. Box 16369, Jacksonville, FL 32245-6369
Office: 11201 Saint Johns Industrial Pkwy S., Jacksonville, FL 32246-7652
Tel. 904.641.7746 · 800.625.1557
Fax 904.641.7378
usa@klsmartin.com · www.klsmartinnorthamerica.com

91-600-07-48 · REV 02 · 2023.06 · IPS Gate Onboarding Flyer · The information presented is intended to demonstrate a KLS-Martin L.P. product. Please refer to the IFU for the full list of indications, contraindications, warnings, precautions, and sterilization information. Always refer to the IFU before using any KLS-Martin L.P. product. Surgeons must always rely on their own clinical judgment when deciding which products and techniques to use with their patients.